



 **Genotropin** *MiniQuick*[®]
somatropin (rbe)

A quick guide




This non-promotional material developed and funded by Pfizer UK is intended for UK patients prescribed Genotropin only.

- Fixed dose – inject entire contents
- Reconstitution in 3 steps
- Preservative-free
- Small injection volume
- Can be stored at room temperature for up to six months

1

Genotropin MiniQuick Instructions for use

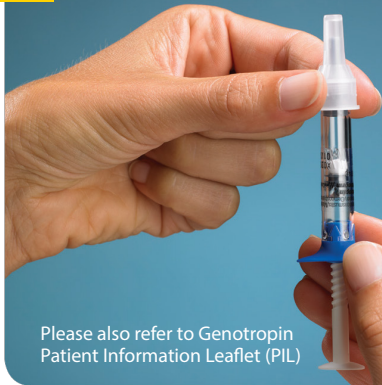

Peel paper covering from needle. Attach needle to Genotropin® MiniQuick by pushing and turning until it stops.

Please make sure the needle is positioned **squarely** onto the end of the rubber bung before screwing it to the Genotropin® MiniQuick.

Please also refer to Genotropin Patient Information Leaflet (PIL)


2



Hold Genotropin® MiniQuick with the needle pointing **upwards**. Turn white plunger rod clockwise until it will go no further.
(There is no need to remove air bubbles before injecting, even if some are present.)
Remove the needle protective cap

3

Firmly pinch skin at injection site and push needle into the skin-fold. Push white plunger rod as far as possible and inject entire contents of Genotropin® MiniQuick. When injection is complete, withdraw needle and dispose of the MiniQuick syringe and needle in a sharps bin, as directed by your physician.

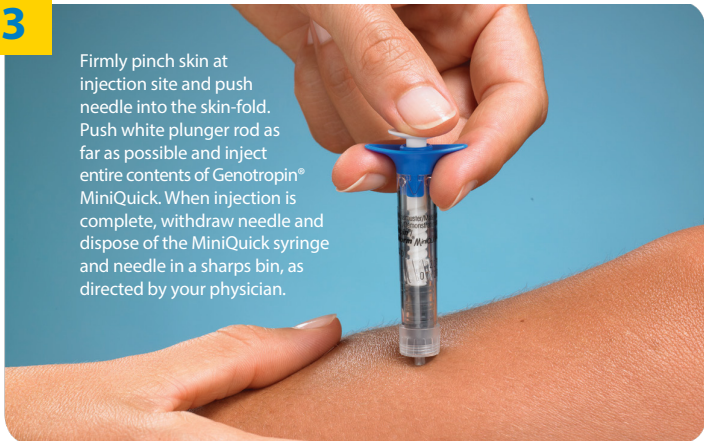


Please note

If any resistance is felt during Stage 2 and 3 it could be the needle has not been positioned correctly on top of the Genotropin® MiniQuick. This will result in the needle being inserted at an angle inside the rubber stopper and a high pressure will have built up inside the cartridge as the air cannot escape.

To rectify this, carefully replace the outer needle cap (opaque white one) over the needle and twist anti-clockwise to remove the needle.

Holding the Genotropin® MiniQuick syringe vertically (needle end upwards) reposition the needle squarely on top of the syringe (if the needle is not pointing upwards, the release of pressure inside the cartridge could result in the solution being expelled instead of the air).

If necessary, apply a new needle to the Genotropin® MiniQuick.

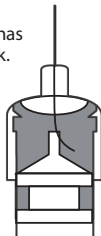

Reporting of side effects

If you get any side effects, talk to your doctor, pharmacist or nurse. This includes any possible side effects not listed in the package leaflet. You can also report side effects directly via the Yellow Card Scheme at <https://yellowcard.mhra.gov.uk> or search for MHRA Yellow Card in the Google Play or Apple App Store.

By reporting side effects you can help provide more information on the safety of this medicine.



PP-GEN-GBR-1975

Date of preparation: April 2025

Walton Oaks
Dorking Road
Tadworth
Surrey KT20
7NS, UK



Your Genotropin®
Experience